

Avoid drug screening hassles

*Think Outside the Cup*



**Lab-based oral fluid drug screening—  
a smart alternative for substance abuse testing**

- Eliminate gender issues
- Prevent tampering
- Oral fluid convenience
- Laboratory accuracy



*The True Standard*

**Intercept<sup>®</sup>**

**Oral Fluid Drug Screening**

## THINK OUTSIDE THE CUP

# With Oral Fluid Convenience

### Swab



1. Place pad between cheek and gums for 3 minutes

### Snap



1. Insert pad in vial
2. Snap wand at scoring

### Seal



1. Recap and apply seal
2. Send to laboratory

**“Oral fluid testing eliminates the gender issues for sample collection processing, which allows the assigned (officer) to collect needed samples from all offenders on their caseloads.”**

*Wayne Cain, Community Corrections Supervisor, Washington Department of Corrections*


### INTERCEPT®—Comparable Positive Rates to Urine Testing

NIDA-5 Drug Panel	Urine Analysis	INTERCEPT®	INTERCEPT® Cut Offs
<b>Overall positive rate</b>	<b>4.46%</b>	<b>5.06%</b>	
Marijuana	3.17%	3.22%	1 ng/mL
Cocaine	0.69%	1.12%	5 ng/mL
Amphetamines	0.29%	0.47%	see below
Opiates	0.29%	0.23%	10 ng/mL
PCP	0.02%	0.03%	1 ng/mL

*Journal of Analytical Toxicology, Vol. 26, December 2002.*

INTERCEPT® amphetamine detection includes assays for amphetamine (100 ng/mL) and methamphetamine (40 ng/mL) **AND ALSO DETECTS ECSTASY**. Additional INTERCEPT® assays: barbiturates, benzodiazepines, and methadone.

THC Positives Detected by Intercept®

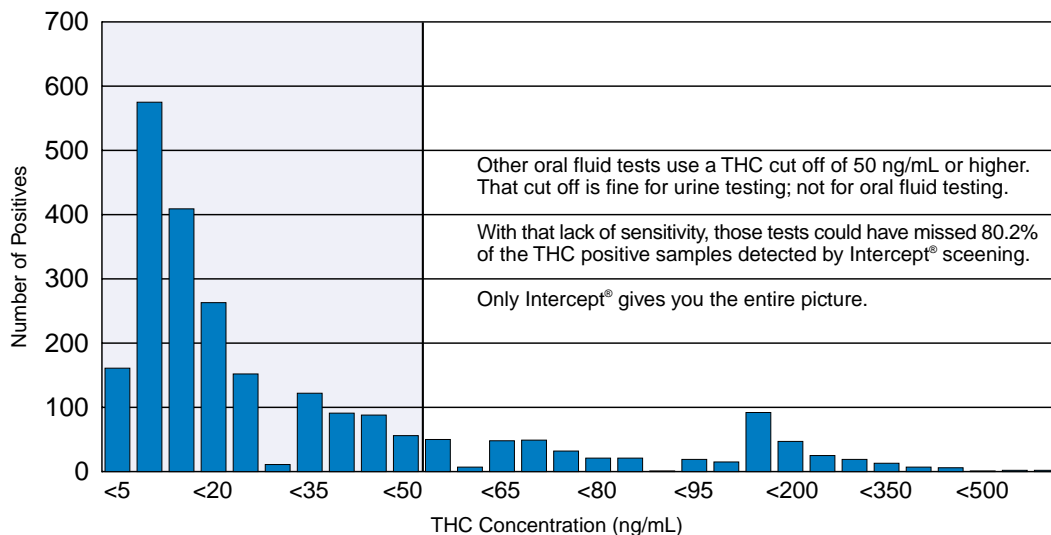

# THINK OUTSIDE THE CUP

## *To Eliminate Collection Issues*



### INTERCEPT®—Give Up Collection Hassles, Not Reliability

#### Eliminates Gender Issues

- Non-invasive collection
- No bathroom required
- Avoid scheduling inconveniences of urine collection

#### Oral Fluid Convenience

- Speeds up and controls collection
- More dignified collection process
- Collection can be done virtually anytime, anywhere

#### Laboratory Accuracy

- Proven laboratory process, comparable to urine testing
- NIDA-5, plus Ecstasy, Benzos, Barbs, Methadone
- 24-hour reporting on negative results

#### Prevent Tampering

- Direct observation prevents sample tampering
- Reduces opportunity for use of adulteration products
- Certified laboratories follow chain of custody

**“Intercept is more sensitive to patient’s needs by being less embarrassing and easier for patients to provide an oral fluid sample compared to that of urine.”**

*Dr. Beny Primm, Executive Director, Addiction & Research Treatment Corp, New York City*

### INTERCEPT®—Laboratory Confidence With Oral Fluid Convenience

	INTERCEPT®	Urine
No bathroom required for collection	Yes	No
Can be collected anytime, without gender issues	Yes	No
Prevents dilution or use of masking agents	Yes	No
Detects NIDA-5, Ecstasy, commonly abused drugs	Yes	Yes
Certified laboratory analysis	Yes	Yes
Training and certification for collection personnel	<1 hour, CD-based, very easy	1-day seminar, many variables on diluted, adulterated or substituted samples

**Ask about our training program** for all staff involved in the collection process to ensure proper procedures.

*To learn more—call 1-800-869-3538 or visit us online at [www.4intercept.com](http://www.4intercept.com).*



# Intercept®

## Oral Fluid Drug Screening

INTERCEPT® tests are available from a national network of certified laboratories.



## **INTERCEPT® — Gives You Control**

### **Think outside the cup with oral fluid convenience**

- Eliminate gender issues
- Simple oral fluid collection
- Laboratory accuracy
- Prevent tampering



### **Simple Collection Procedure ... Swab, Snap, Seal**

Donor's mouth must be clear for 10 minutes prior to providing sample.

Place collection pad between lower cheek and gums and gently rub back and forth until the pad is moist.  
*Once moist, leave collection pad between cheek and gums for three (3) minutes.*

After three minutes, open specimen vial in an upright position by gently rocking the cap back and forth

Push collection pad into specimen vial as far as it will go.

Snap collection wand at scored line against side of vial. Place cap onto vial until it snaps.  
*Do not tilt vial or spill fluid inside.*

Place tamper evident seal across top of specimen vial if required.

Seal Intercept® specimen vial(s) and chain of custody form into a specimen bag for shipment to laboratory.

**“We find the ability to test quickly and to collect the sample with minimal exposure to staff is a great advantage.”**

*Janet Ward, Drug Court Coordinator, Anne Arundel County (MD) District Court*

**To learn more—call 1-800-869-3538 or visit us online at [www.4intercept.com](http://www.4intercept.com).**



**OraSure Technologies, Inc.**  
diagnostic solutions for the new millennium

©2003 OraSure Technologies, Inc. Pending US patents and various international patents.  
INTERCEPT is a registered trademark of OraSure Technologies, Inc. Item# L3001-1035 (Rev. 08/03)



**Intercept®**  
**Oral Fluid Drug Screening**